


ZSI 100 KIT

ZSI 100 Kit comes with two adjustable rods of 25 cm.

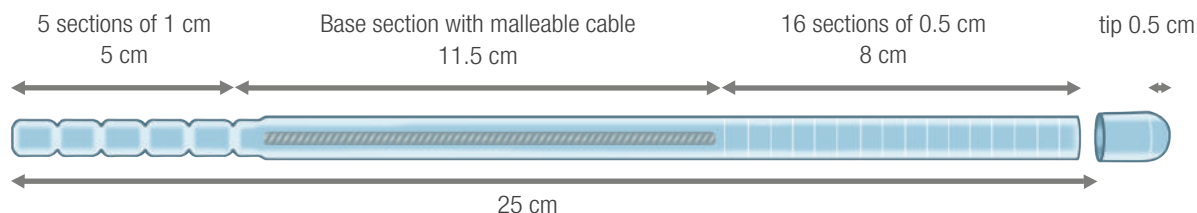

ZSI 100 MEASURER

Surgeon should request a measurer from the distributor, which is not included in the ZSI 100 Kit.




+++ TO ADJUST THE LENGTH, ALWAYS CUT THE DISTAL PART FIRST +++

From 25 cm to 17 cm cut the distal part only, never the proximal part first.


SIZE LOWER FROM 25 cm to 17 cm

EXAMPLE 1: ZSI 100 of 19 cm: measure 18.5 cm from the proximal part, (1) cut the distal part and add the tip of 0.5 cm = 19 cm.

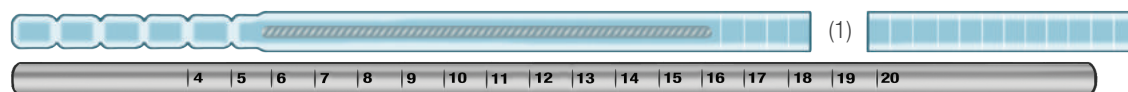

Example 18.5 cm + cup = 19 cm


EXAMPLE 2: ZSI 100 of 17 cm: measure 16.5 cm from the proximal part, (1) cut the distal part and add the tip of 0.5 cm = 17 cm.

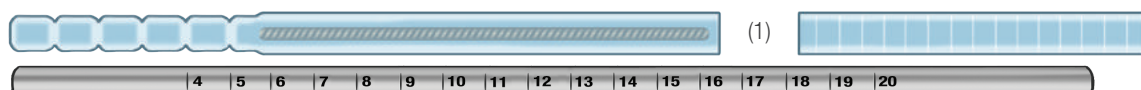

Example 16.5 cm + cup = 17 cm


SIZE LOWER THAN 17 cm

EXAMPLE 1: ZSI 100 of 14 cm: (1) cut the distal part first, measuring 13.5 cm from the distal end, and (2) cut the proximal part and add the tip of 0.5 cm = 14 cm.

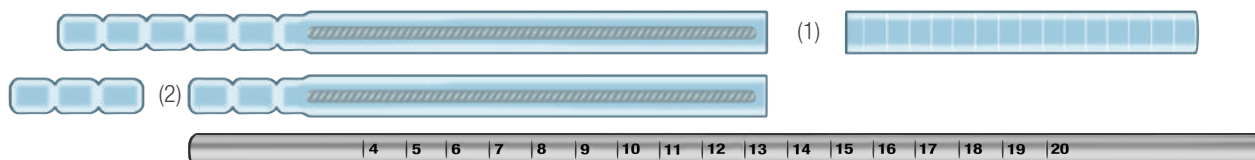

Example 13.5 cm + cup = 14 cm

EXAMPLE 2: ZSI 100 of 12 cm: (1) cut the distal part first, measuring 11.5 cm from the distal end, and (2) cut the proximal part and add the tip of 0.5 cm = 12 cm.

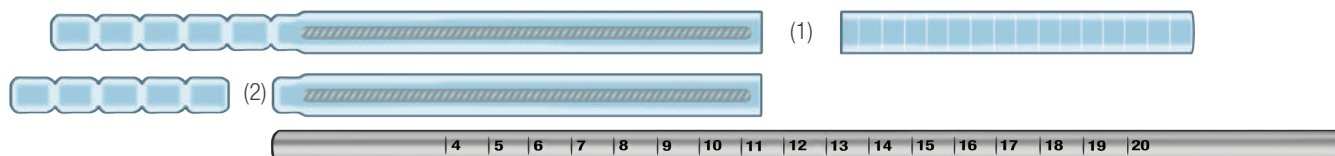

Example 11.5 cm + cup = 12 cm


ZSI 100 PREPARATION MANUAL

ZSI 100 Kit comes with an adjustable 25 cm device.

Example preparation ZSI 100 25 cm




* Connect the cup when the device is dry. Then you can immerse it in antibiotics.

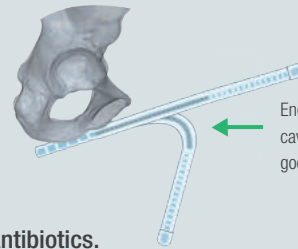

Enough cable in the distal corpus cavernosum to bend the device with good memory shape

Example preparation ZSI 100 19 cm

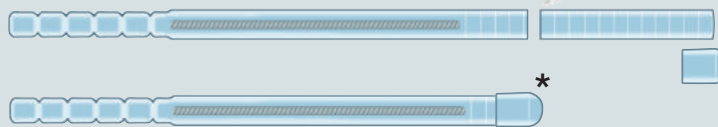


* Connect the cup when the device is dry. Then you can immerse it in antibiotics.



Enough cable in the distal corpus cavernosum to bend the device with good memory shape

Example preparation ZSI 100 17 cm

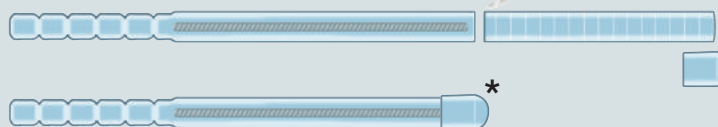

* Connect the cup when the device is dry. Then you can immerse it in antibiotics.



Enough cable in the distal corpus cavernosum to bend the device with good memory shape

Example preparation ZSI 100 12 cm




* Connect the cup when the device is dry. Then you can immerse it in antibiotics.




Enough cable in the distal corpus cavernosum to bend the device with good memory shape

CUT DISTAL PART FIRST AND IF NEEDED CUT PROXIMAL PART TO ADJUST THE LENGTH

Example of a bad preparation ZSI 100

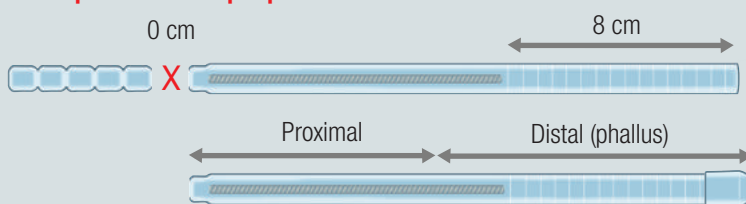

DO NOT CUT ONLY PROXIMAL PART

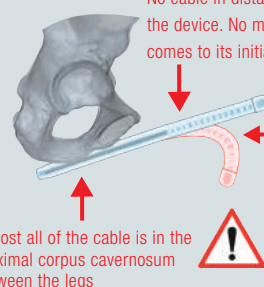

No cable in distal corpus cavernosum (phallus) to bend the device. No memory shape in the phallus. Silicone comes to its initial position.

No cable in distal corpus cavernosum (phallus) to bend the device. No memory shape in the phallus. Silicone comes to its initial position.