

15 chronological steps that should be followed and performed when implanting the ZSI 375.

1. INSERT 16 FOLEY CATHETER ☐
2. PERFORM DISSECTION ☐
3. PREPARE THE DEVICE, OR, TAKE THE PRE-FILLED ZSI 375 ☐
4. DEACTIVATE THE DEVICE ☐
5. IMPLANT THE CUFF AS TIGHT AS YOU SEE FIT ☐
6. SUTURE THE CUFF ☐
7. REMOVE 16 FOLEY CATHETER ☐
8. ACTIVATE THE DEVICE ☐
9. CHECK SPRING POSITION. ADJUST SALINE VOLUME AS NEEDED ☐
10. DEACTIVATE THE DEVICE ☐
11. INSERT THE PUMP ☐
12. INSERT A 12 FOLEY CATHETER FOR 24 HOURS ☐
13. SUTURE ☐
14. STICK TRACEABILITY STICKER ON PATIENT FORM, INCLUDED IN THE BLISTER ☐
15. SEND BACK THE COMPLETED PATIENT FORM WITH TRACEABILITY STICKER ☐

This is a complementary checklist to the steps as discussed in the
Operating Room Manual